

FABRIC BACKREST MANUAL FOR ROLLATORS



Contents

[1.- Important](#)

[2.- Icons used on the label and in the manual](#)

[3.- Product Indications](#)

[4.- Product Description](#)

[5.- Assembly and Use](#)

[6.- Manufacturer Data](#)








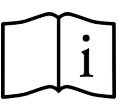


1.- Important

Read this manual carefully before using this product and keep it for future reference if needed.

If you cannot read or understand any warning, caution, or instruction, please contact the seller before using this product, as otherwise there may be risks and damage.

Consult your physician or orthopedist before using this product.

2.- Icons used on the label and in the manual

	Manufacturer details
	Date of manufacture
	Medical Device
	Observe safety instructions
	Maximum user weight in kg
	Declaration of Conformity
	Caution, danger of entrapment
	Check the user manual
	Product recycling information
	Recycling centre
SN	Product serial number
LOT	Product batch number
REF	Product reference or code

3.- Product Indications

The fabric backrest for rollators is an accessory for Forta brand walkers; it cannot be placed on any models other than those specified below, nor on those from other manufacturers. This accessory consists of padded and reinforced fabric sewn onto plastic pieces that attach

to certain Forta walkers. The backrest must be properly placed to ensure its correct functioning. This backrest can be attached to the following walkers:

- Drakon 1
- Drakon 2
- Kanguro
- Kanguro HD
- Kanguro F1
- Kanguro F2
- Kanguro Plus
- Komodo 1
- Komodo 2
- ARA-Plus
- ARA-D

4.- Product Description




1. Padded and reinforced backrest fabric
2. Backrest anchor

5.- Assembly and Use


To mount the fabric backrest on one of the compatible walker models mentioned above, follow these steps:

1. Place the anchor next to the corresponding part on the walker. The part is located directly below the handle's adjustment clip and has a hole on each of its sides.
2. While holding the walker by the handle with one hand, use the other hand to press the anchor on one side and try to fit the pin (the protrusion) into the hole in the walker part. You can move the anchor until you feel that side click and the pin goes into the hole.
3. Once one side of the anchor is engaged, press on the opposite side to fit it. Make sure both pins are properly inserted into the hole and that the anchor fully wraps the walker.

4. Make sure the anchor is secured by pulling it outward firmly. If the anchor does not come loose, repeat the previous steps on the anchor on the opposite side.
5. Once both anchors are secured, pull on the backrest while holding the walker to check that it does not come loose, ensuring it is properly attached.

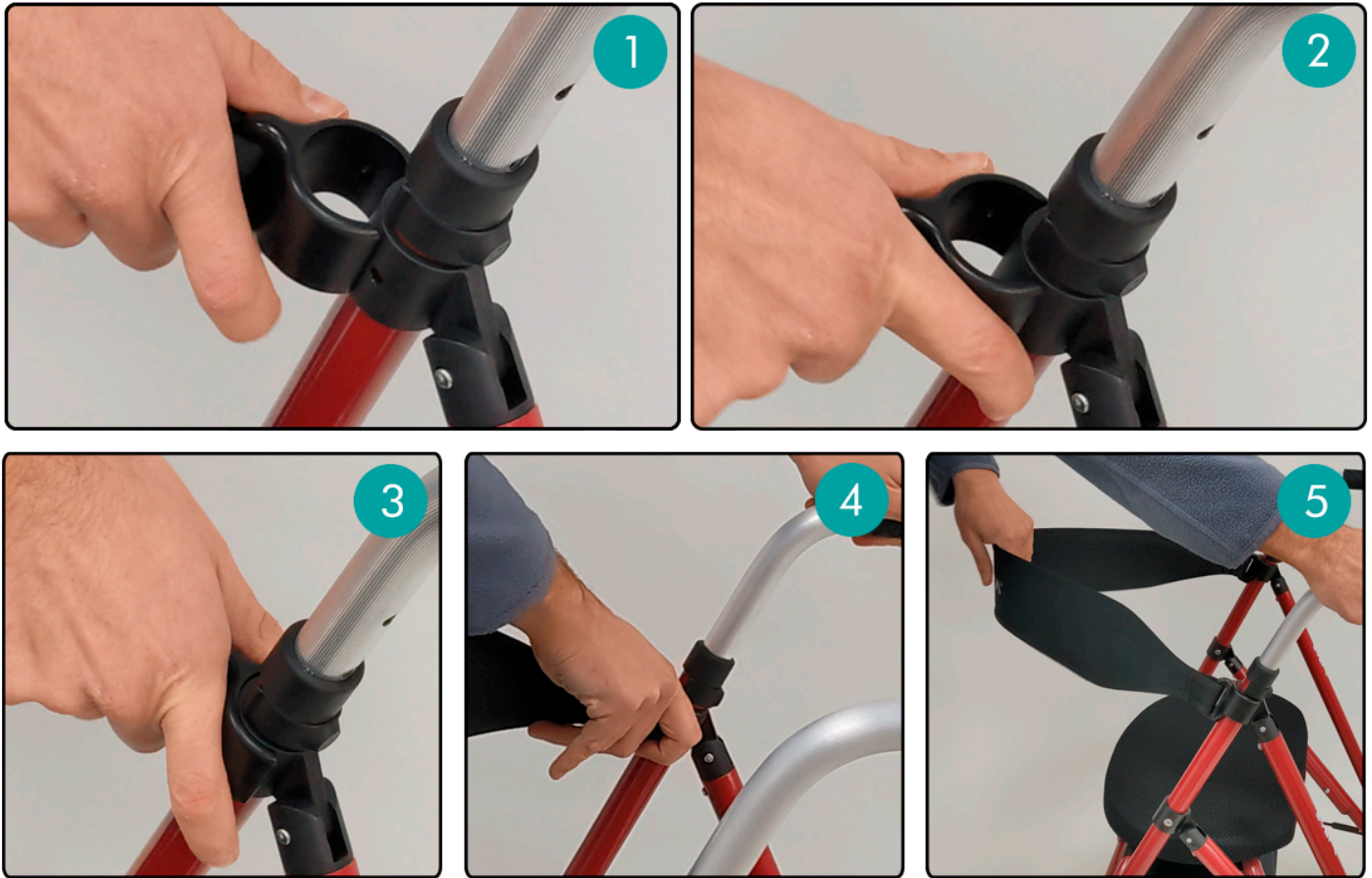

If the anchors are not properly secured, the backrest may come loose and the user could fall when leaning back. You must ensure that both pins on each anchor are well inserted into the walker. Before sitting in the walker and using the backrest, make sure both anchors are fully secured. Pull each one firmly and pull the backrest fabric outward to verify.

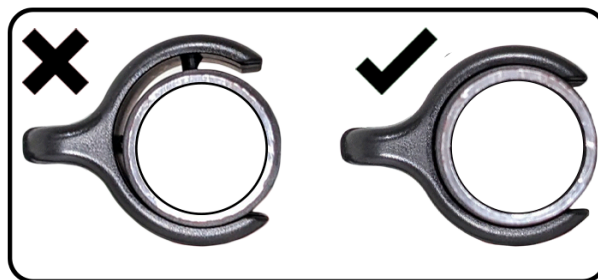

If you need to disassemble the backrest, it is advisable to use a flat-head screwdriver. Insert the screwdriver tip on one side of the anchor and turn it to release that side. Once one side is free, simply pull the anchor to remove it completely.

6.- Manufacturer Data

FORTA - FABRICACIONES ORTOPÉDICAS ALBACETE, S.L.

Pq. Emp. Campollano, c/C 90 Nave 1

02007 Albacete

España - Spain

Apartado de Correos 5316

info@fortasl.es

www.fortasl.es



MADE IN EUROPE



FABRICACIONES ORTOPÉDICAS ALBACETE, S.L.